**X-RAY TUBES
PRODUCTION**

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TECHNICAL DATA

Voltage	nominal	125	kV
	maximum for test	137	
Inverse voltage	nominal	135	kV
	maximum for test	140	
Focal spots	small	2.2 x 2.2	mm
	large	3.8 x 3.8	
Filament characteristics	small	3.0 ÷ 6.0 V 3.0 ÷ 5.5 A	
	large	4.0 ÷ 12.0 V 3.0 ÷ 5.5 A	
Anode material		tungsten	
Target angle		15	°
Anode heat storage capacity		55000	J
Maximum anode cooling rate		380	W
Nominal anode input power at 0.1 s (AC)	small	5000	W
	large	13800	
Inherent filtration		0.5 mm Al	
Maximum diameter		60	mm
Overall length		197	mm

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GENERAL INFORMATION

When mounting tube inserts adopt proper caution, in order to avoid glass bulb breaking and fragments projection. Please use protective gloves and glasses. Tube insert connected to H.V. supply is a radiation source: be sure to take all necessary safety cautions

> Wash thoroughly with alcohol the external surface of tube insert (care of fire risk). Avoid contact of dirty surfaces with cleaned tube insert.

> Clamp system inside housing or self-contained units must not mechanically stress the tube.

> After installation, check the right working of the tube (no fluctuation of tube current nor crackling)

> Comply with insert thermal parameters, planning and programming the exposure parameters and cooling pauses.

> Housing or self-contained units must be provided with an adequate thermic protection.

> Voltages indicated in charts are valid for transformer supplied with ground center.

> It is extremely important to observe the connection diagram and the grid resistor value. Any change could modify the dimensions of the focal spot, also varying diagnostic performances or overloading anode target.

> Tube inserts contain environment polluting materials, particularly lead liner tubes. Please apply to qualified operator for waste disposal, according to local regulation requirements.

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