ENGLISH

TECHNICAL DATA

X-ray tube voltage	<i>nominal</i>	120 kV	IEC 60613:2010
	<i>overvoltage test</i>	132 kV	IEC 60601-2-28:2020
Nominal focal spot		0,5	IEC 60336:2020
Filament characteristics		3,5÷6,0 V	ver. N ²
		2,5÷3,5 A	
		1,8÷4,0 V	ver. P ³
		2,4÷3,6 A	
Anode material		Tungsten	
Target angle		10°	
Anode heat storage capacity¹		40000 J	
Continuous anode input power¹		500 W	IEC 60613:2010
Nominal anode input power 0.1 s – DC		1900 W	ver. N ²
		1820 W	
Permanent filtration		1,2 mm Al	IEC 60522-1:2020
Maximum diameter		42 mm	
Maximum overall length¹		172mm	
Weight (without lead shield)¹		400 g	

¹ with radiator

² Version N: ORDER CODE 2076 XX XX XX XX (XX digits to be defined according to tube configuration).

³ Version P: ORDER CODE 2176 XX XX XX XX (XX digits to be defined according to tube configuration).

This model is designed for improved image resolution in CBCT applications.

See following pages for differences in filament, emission and rating charts between the two version.

WARNINGS FOR THE USER

When mounting tube inserts adopt proper caution, in order to avoid glass bulb breaking and fragments projection. Please use protective gloves and glasses.

Tube insert connected to H.V. supply is a radiation source: be sure to take all necessary safety cautions

- Wash thoroughly with alcohol the external surface of tube insert (care of fire risk). Remove any labels on the glass bulb. Avoid contact of dirty surfaces with cleaned tube insert.
- Clamp system inside housing or self-contained units must not mechanically stress the tube.
- After installation, check the right working of the tube (no fluctuation of tube current nor crackling)
- Comply with insert thermal parameters, planning and programming the exposure parameters and cooling pauses. Housing or self-contained units must be provided with an adequate thermic protection.
- Voltages indicated in charts are valid for transformer supplied with ground center.
- Tube inserts contain environment polluting materials, particularly lead liner tubes. Please apply to qualified operator for waste disposal, according to local regulation requirements.

INCIDENT REPORT ACCORDING TO MEDICAL DEVICE REGULATION 2017/745 AS AMENDED

In order to comply with CE marking requirements, end users have to report to local Competent Authority all the information about possible incidents involving the device, regarding any deterioration in its characteristics and performances, as well as any inaccuracies in this documentation, which might lead to or might have led to the death of patient / user or a deterioration in his state of health. This information must be promptly reported also to C.E.I. in order to start manufacturer reporting, as per above mentioned Regulation.

This mark assures device conformity to
Medical Device Regulation 2017/745 as amended.

INFORMATION:

Even if the tube is shipped after a long seasoning and testing cycle, it is always better to perform a quick seasoning before the first exposure. Please start the test cycle with the lowest kV possible in your machine, and increase the kV gradually with an incremental step of 5kV. For each step, perform at least 3 exposures maintaining the current parameter between 1mA and 2mA. A gap of few seconds between each of the exposures is advised. Increase the kV to the max kV as rated for the respective tube. In case of instability observed, reduce kV by one step. Once the kV reaches the nominal kV allowed for the respective tube, perform 3 exposures at the nominal power as requested for the concerned application.

Full seasoning is not required, when installing the machine. Nevertheless, if the tube head has been stored for more than 6 months, it is advised to perform few exposures at low kV and low power, before sending the machine to the final customer.

Sometimes a gentle metallic noise is audible from inside the tube. This noise is not affecting tube performances and the use of such tubes is totally safe.

Kindly note that power rating has been verified in a large oil tank. Power rating may vary depending upon the oil capacity as well as the oil circulation in the testing tank.

Kindly note that emission charts indicate typical characteristics. Emission charts have a tolerance up to $\pm 10\%$ related to filament current.

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